

Supplies needed:

Sewing machine, Scissors

8.5" x 8.5" cloth squares (100% cotton or muslin, not stretchy)

Use 2-4 layers, depending on thickness of cloth

Two 7" pieces of elastic (1mm-2mm thickness works well)

DROP OFF INSTRUCTIONS:

11400 W. Lake Park Dr. Milwaukee - Leave at the back loading zone

Contact: Marcy at (414) 788-8299, marcy.stutzman@hhch.net

CLOTH SURGICAL MASK SEWING INSTRUCTIONS

- 1 Line up 3 layers of fabric squares (8.5" X 8.5")**
- 2 Sew around the square on 3 sides, 1/4" from the edge (be sure to catch all the layers)**
- 3 Flip inside out (like a pillowcase)**
- 4 Fold the 4th side in and sew that edge (pin if needed)**
- 5 Make 3 pleats starting at the top (pin if needed)
Sew on the right and left edges to keep the pleats in place**
- 6 Sew 7 inches of elastic on each side of the mask, sew at the top and bottom (go back and forth a few times so it will stay). Leave a 1/4" seam allowance for the elastic**

